



Imperial Fastener Company, Inc.

Cubicle Tracks • Cubicle Curtains • Intravenous Support Systems

Polyester Curtain Care Instructions

Laundering:

Machine wash in water not to exceed 160°F, using Synthetic setting and mild detergent. Do not use bleach or fabric softener. Do not extract 30 second spin cycle may be used to remove excess moisture. Remove load immediately.

Drying:

Tumble dry 3-5 minutes on Synthetic cycle, not to exceed 100°F, until damp dry and remove immediately.

Finishing:

No finishing is required if fabric is re-hung immediately following drying cycle. Light touch-up with a hand iron, not to exceed 275°F, may be desired.